




RAW-23140 REV 00



**Tactical Black
PN 30-0001**

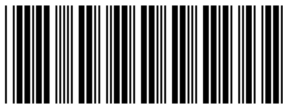

NSN 6515-01-521-7976

U.S. Patent No. 7,842,067; 7,892,253;
Patents pending in other foreign countries
Licensed and Manufactured by:
Composite Resources, Inc. 803.366.9700

WARNING: THIS IS A SINGLE USE PRODUCT

This product is intended for qualified trained professionals in the application of a tourniquet. It is not intended for the general public. Use only as directed by your EMS authority or under the supervision of a physician. Read entire Instructions For Use prior to using this product. Inappropriate application can result in serious injury or loss of limb. The use of any tourniquet is **A LAST RESORT** and should only be employed when bleeding cannot be stopped and the situation is life threatening.

LIMITATION OF LIABILITY:

Composite Resources, Inc., its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application and use of the Combat Application Tourniquet® (C-A-T®). The user assumes all risk of liability.

Composite Resources, Inc.
485 Lakeshore Parkway
Rock Hill, SC 29730

EC

REP



Mdi Europa GmbH
Langenhagener Straße 71
D-30855 Langenhagener

Combat Application Tourniquet®

Instructions for Use: Two-handed Application

To prepare for use, store the C-A-T® in its one-handed configuration



1 Apply tourniquet proximal to the bleeding site. Route the band around the limb and pass the red tip through the inside slit of the buckle. Pull the band tight.



2 Pass the red tip through the outside slit of the buckle. The friction buckle will lock the band in place.



3 Pull the band very tight and securely fasten the band back on itself.



4 Twist the rod **until bright red bleeding has stopped and the distal pulse is eliminated.**



5 Place the rod inside the clip; locking it in place. **Check for bleeding and distal pulse.** If bleeding is not controlled, consider additional tightening or applying a second tourniquet proximal side by side to the first and reassess.



6 Secure the rod inside the clip with the strap. **Prepare the patient for transport and reassess.** Record the time of application.

Instructions for Use: One-handed Application



1 Apply the tourniquet proximal to the bleeding site. Insert the wounded limb through the loop formed by the band.



2 Pull the band **very tight** and securely fasten the band back on itself.



3 Adhere the band around the limb. Do not adhere the band past the rod clip.



4 Twist the rod until **bright red bleeding has stopped and the distal pulse is eliminated.**



5 Place the rod inside the clip locking it in place. **Check for bleeding and distal pulse.** If bleeding is not controlled, consider additional tightening or applying a second tourniquet proximal side by side to the first and reassess.



6 Adhere the band over the rod, inside the clip, and fully around the limb.



7 Secure the rod and band with the strap. **Prepare for transport and reassess. If possible, record time of application on white strap.**

Storing in the One-Handed Configuration

To prepare for use, store the C-A-T® in its one-handed configuration



1 Pass the red tip through the inside slit in the buckle. Pull 6" of band through, fold it back and adhere the band to itself.



2 Flatten the loop formed by the band. Place the buckle in the middle of the flattened band.



3 Fold the C-A-T® in half placing the buckle at one end. The C-A-T® is now ready to be placed in your medical kit.

EMS Orange
NSN Pending



PN 30-0023



Exclusive distribution in the U.S. by:
North American Rescue®, LLC
1-888-689-6277

www.NARescue.com info@NARescue.com



Trainer Blue
NSN 6910-01-560-2972



PN 30-0033