

Raptor™ IV Securing Device

PN: 30-0006 NSN: 6515-01-523-3317

Instructions for Use

IV Securing Device

1. Remove white protective film.
2. Place tubing or saline lock on adhesive surface.
3. Fold adhesive surface over the IV tubing or saline lock and secure to the upper extremity.
4. Ensure securing device does not impede arterial or venous blood flow once applied.

NOTE: Use only as directed by your EMS authority or under the supervision of a physician.

REV. 01.13.10 Made in Singapore



NORTH AMERICAN RESCUE®

35 Tedwall Ct. • Greer, South Carolina 29650

1.888.689.6277 www.NARescue.com

NAR Item Number



30-0006

Quantity



6 EACH