Hypothermia Prevention & Management Kit™ (HPMK®)

Hypothermia Prevention & Management Kit - Insulated™ (HPMK-IT™)

 Translations

REV021723 ZZ-1500

ENGLISH

1. Open Kit by tearing at the Red-Notched Markings.
2. Remove the NAR Ready Heat™ Blanket from its packaging to allow it to begin warming. The Liner is activated when it comes in contact with air.
3. Remove any wet clothing from the casualty and replace with dry clothes if possible.
4. Remove the Heat Reflective Shell (HRS or HRS-I) from the kit and place casualty inside. Place the hood in position over the casualty's head.
5. Place the NAR Ready Heat™ Blanket on top of casualty.

WARNING- The Blanket should not come into direct contact with the casualty's skin.

6. Wrap and secure the Heat Reflective Shell (HRS or HRS-I) around the casualty. This will effectively retain the heat generated by the NAR Ready Heat™ Blanket.
7. Continue to monitor the casualty during evacuation. The casualty can be reassessed by opening the Heat Reflective Shell (HRS or HRS-I) to evaluate injuries or interventions.

 Lot Number   Authorized Representative  Single Use  Manufacturer

 Not Made with Natural Rubber Latex  Not for Use in MRI  Device Part Number  Use by Date  Consult Instructions for Use

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