

- 1** Shake Field Shield bottle well before use. Clean & disinfect area if necessary. ✓




- 2** Press pump firmly to apply mist, repeating until area is fully covered. ✓



- 3** Dab around area to remove excess solution. **DO NOT RUB IN.** ✓



- 4** Protection begins once the solution has set into thin film (typically within 1-2 minutes). ✓

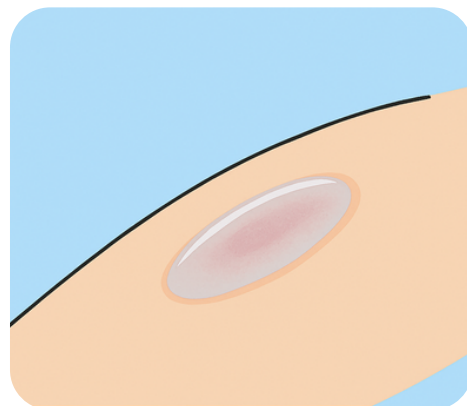

- 5** Wipe or scrape nozzle surface prior to and after each use to remove film & prevent clogging. ✓




- 6** Reapply every 24-72 hours by following steps 1-4. Remove if necessary by cleaning or debriding. ✓



OPTIONAL:

Use a secondary dressing if necessary for exudate management or for additional protection of the wound, especially in high impact areas. Secondary dressing can be applied once Field Shield has set.

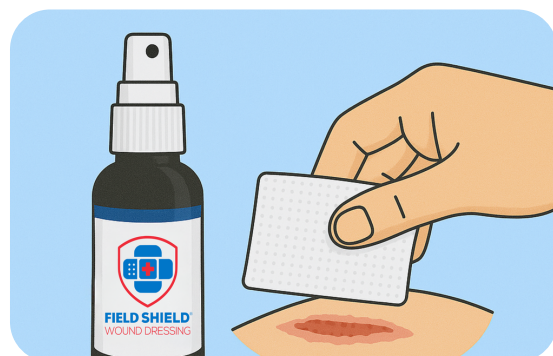

Field Shield is proudly
Made in the USA



FIELD SHIELD®
WOUND DRESSING

MILITARY GRADE

Antimicrobial Wound Care
with Silver Hydrosol and Pain
Relieving Lidocaine

NSN 6510-01-727-1966
(1oz spray)


NSN 6510-01-727-1987
(0.5oz spray)



MARINE FACIAL LACERATION



Day 1 Post Injury

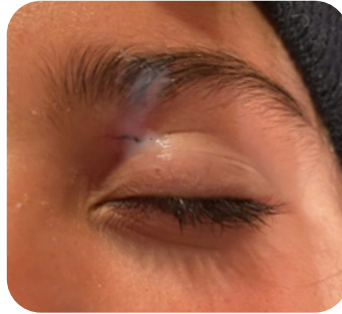

Day 3 with Treatment Applied



Fully Healed



BROWN RECLUSE SPIDER BITE



7 Days - No Treatment



1 Day Post Treatment



5 Days Post Treatment



Fully Healed

MOTORCYCLE ACCIDENT



Initial Injury



1 Day Post Treatment



1 Week Post Treatment



Fully Healed after 2 Weeks

TRAINING COMPETITION INJURY




Initial Injury



Post Treatment



1 Week Partially Healed

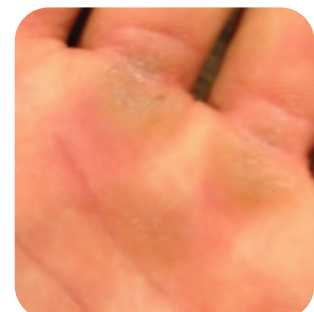

Fully Healed

RIBCAGE ROAD RASH



Initial Injury



3 Days Post Treatment



10 Days Post Treatment



Fully Healed

ROPE BURN



Initial Injury



Post Treatment



Partially Healed



Fully Healed

PARTIAL THICKNESS CONTACT BURN




Initial Burn



Day 1 Debrided



1 Week with Treatment

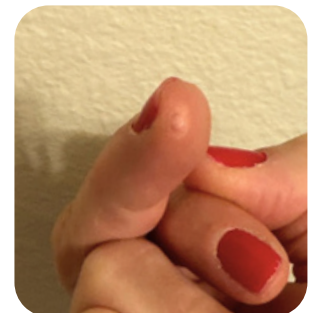

Fully Healed after 23 Days

SHIN PUNCTURE WOUND



Initial Injury



Post Treatment



Fully Healed

MILITARY GRADE



**Antimicrobial Wound Care with Silver Hydrosol
and Pain Relieving Lidocaine**

NSN 6510-01-727-1966 (1oz spray)
NSN 6510-01-727-1987 (0.5oz spray)



Made in the USA

