



# MARK IV ICU Bed

PN: 66-0019



The Mark IV Field Hospital Bed is designed for rapid deployment by single person. With minimal training, it can be set up in less than 60 seconds. Constructed from a bi-folding Altrunium™ material frame, the bed includes a polyurethane foam pad with a water resistant decontaminable cover. The bed is manufactured to support a resting force of 850lbs without material fatigue, and is extremely compact for easy transportation and storage.

The MARK IV Field Hospital Bed can be used in conjunction with mobile shelters / disaster relief applications.

## INSTRUCTIONS FOR USE



1




OPEN THE VELCRO.

2



PREPARE THE BED IN THE FLAT STATE.

3

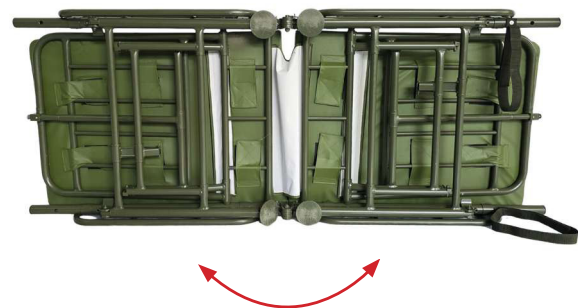

EXTEND POLES OF BED TO 180°.




# MARK IV ICU Bed

PN: 66-0019

## INSTRUCTIONS FOR USE (CONTINUED)

4

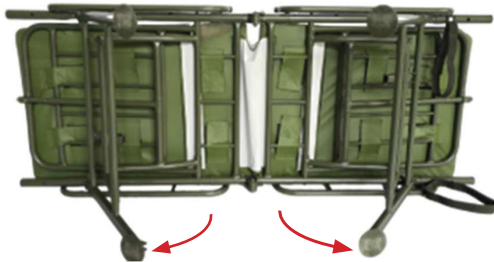

Fold out either side of the support legs until hinge is fully extended.

5



Place the bed upright and make sure the feet lay flat. Press button to adjust sides of bed up and down

6



To raise or lower rail, press both buttons on sides of rail simultaneously.

7



Lock the latch (S/S Lock Plate) to the bed to assure that it does not collapse unexpectedly with uneven weight distribution.

8



At this point, the bed can be fitted with the Meal Tray, IV Pole, and Patient Chart.



**NORTH AMERICAN RESCUE®**

35 TEDWALL COURT • GREER, SOUTH CAROLINA 29650  
PHONE: (864) 675-9800 • FAX: (864) 675-9880  
WWW.NARESCUE.COM • 1-888-689-6277

Revised 061126