

33" Litter Stands

33" Litter Stands: PN:60-0014 NSN:6530-01-523-3314 CE

INSTRUCTIONS FOR USE


● **STEP 1:**

To open, unlatch Velcro® strap that holds bottom rails of litter stand together during transport.



● **STEP 2:**

Pull legs apart and ensure the crossbar supports locks into place.

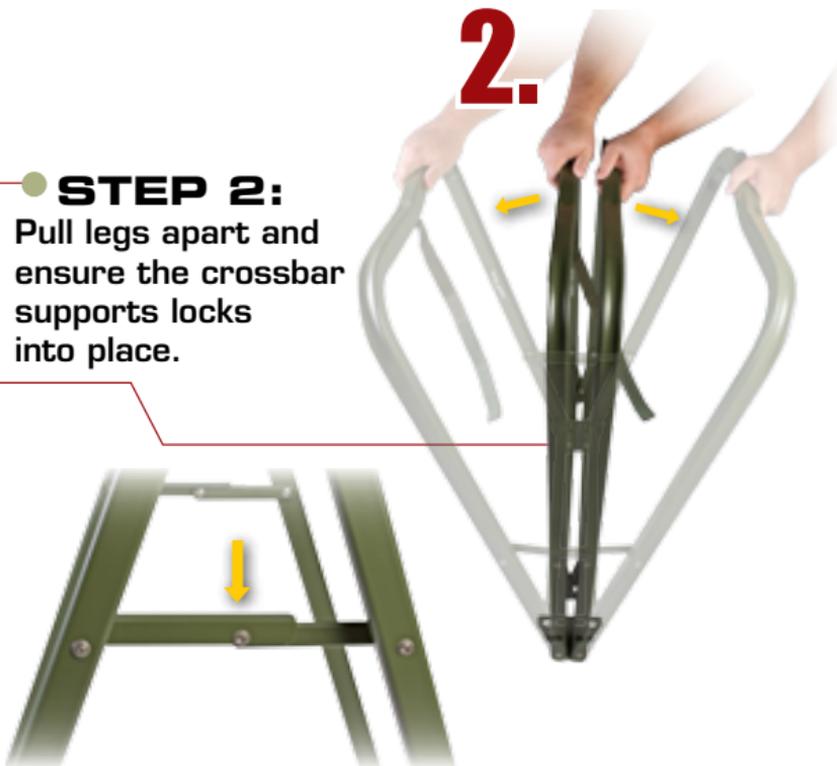

PRODUCTS WITH A MISSION®



NORTH AMERICAN RESCUE®


www.NARescue.com • Greer, SC 29650 • 888.689.6277

33" Litter Stands

33" Litter Stands: PN:60-0014 NSN:6530-01-523-3314 CE

(INSTRUCTIONS FOR USE CONTINUED)

- **STEP 3:**
Turn litter stand over and place bottom rails on floor

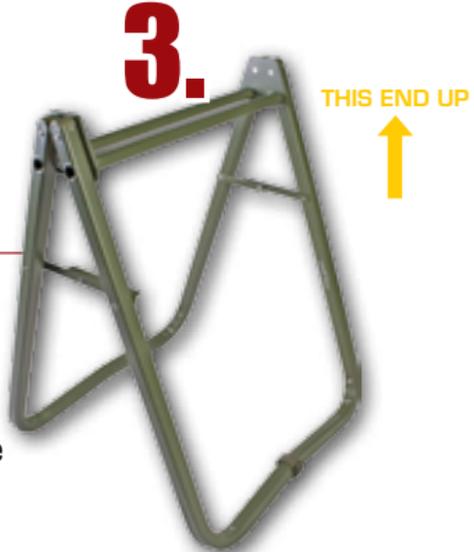

- **STEP 4:**
When positioning the litter stands, ensure that the stands are separated enough to allow the feet of the litter to set in between the two stands.

NOTE:
To close litter stand, repeat steps 1-3 in reverse order.



NOTE: It is recommended that all litter stanchions should be weighted to provide added stability. (Figure 1)



FIGURE 1

REV.122809



NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277

PRODUCTS WITH A MISSION®