

NAR Litter Backrests

Talon® Litter Backrest: PN: 60-0027 CE

OSL™ Litter Backrest: PN: 60-0024 CE

INSTRUCTIONS FOR USE

- **STEP 1:**
Unsnap the backrest fabric straps and loosen backrest fabric.

NOTE:
This allows for the backrest frame to spread and fit over the sides of the litter.



- **STEP 2:**
Spread backrest frame and attach mounting brackets to litter poles on each side of the litter.

NOTE:
Attachment point to the litter poles should be inside and against the litter feet.

NOTE: For 45° configuration, place backrest on litter as shown above. For 90° configuration flip backrest over and place on litter.

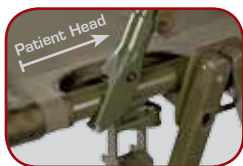


Figure 1-
45° Configuration

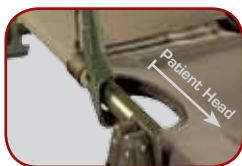


Figure 2-
90° Setup Configuration
(flip backrest over)



NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277

NAR Litter Backrests

Talon® Litter Backrest: PN: 60-0027 CE

OSL™ Litter Backrest: PN: 60-0024 CE

(INSTRUCTIONS FOR USE CONTINUED)

STEP 3:
Tighten backrest fabric by snapping the backrest fabric straps on each side.



45° Configuration

STEP 4:
To remove the backrest, unsnap the backrest fabric straps, loosen the backrest fabric and remove the backrest frame from the litter poles.



90° Configuration

NOTE: The backrest is made to fit snugly onto the OSL™ Litter. Therefore, it may take some effort to completely install the backrest.



Backrest in 45° Configuration



Backrest in 90° Configuration

EC	REP
M3 Europa GmbH Lagerhofweg 51, 71 3850 Haverhorst-Lagerhorst Germany	

PN: ZZ-0292 REV.120110



NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277