


# ENSURE THE FOLLOWING THINGS ARE PRESENT AND IN-ORDER BEFORE DEPLOYING.

It is important to understand that the C-A-T® is a one-time use medical device. Warriors need to take a new tourniquet into battle and ensure that they do not deploy with a C-A-T® Tourniquet that has been used previously or in training.

INSPECTING C-A-T® FOR SERVICEABILITY	PASS	FAIL
1. Start inspection at the Red Tip		
2. Self-Adhering Band		
a.No tearing or fraying of the nylon material (all stitching is intact)		
b.No grime, dirt, or debris in the Self-Adhering Band		
3. Composite Components (there should be no fissures or warping)		
a.Windlass is secured to inner band		
b.Stabilization Plate is intact		
4. Internal Band (there should be no creases indicating prior usage)		

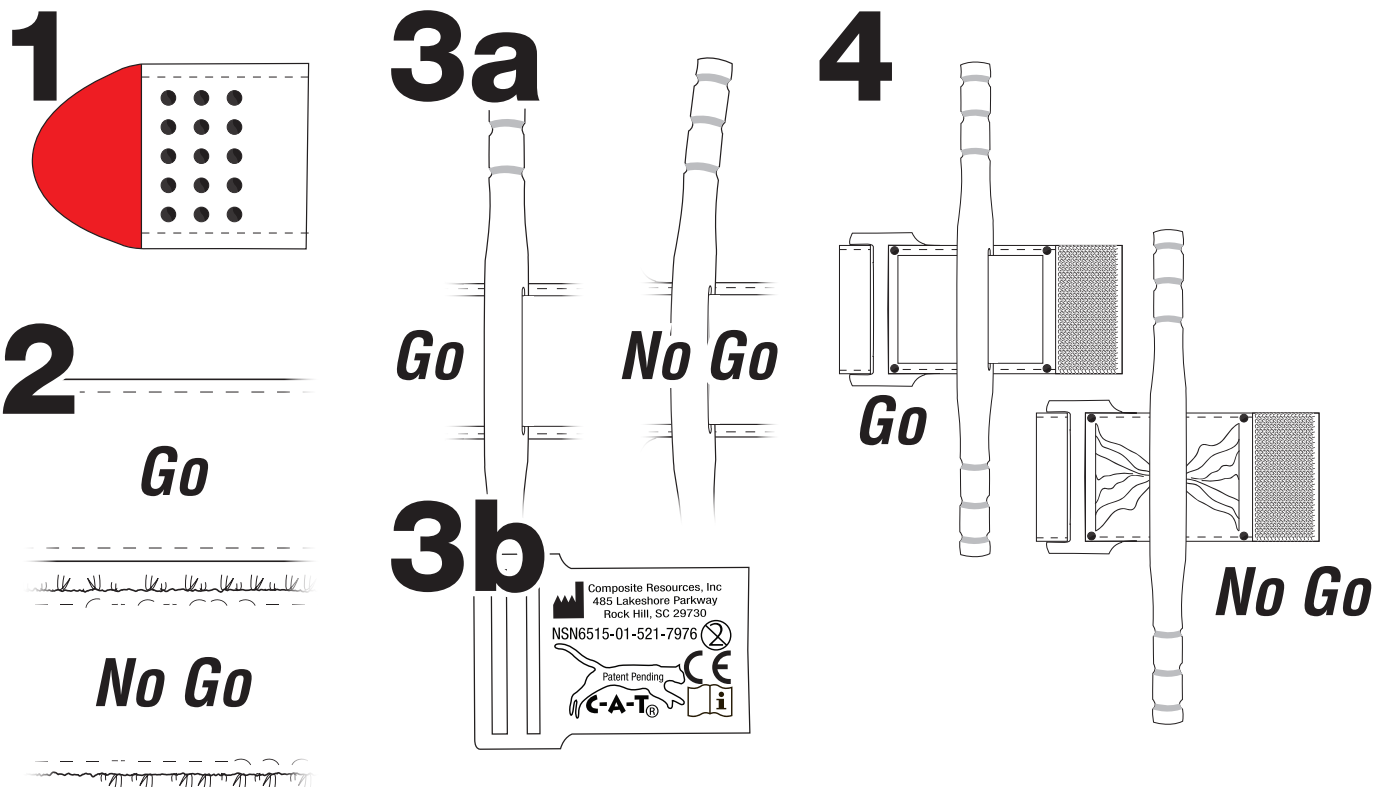

Once inspection is complete return C-A-T® Tourniquet into its ready position by storing the C-A-T® in the one-handed configuration. Please refer to instructions for use for proper technique.

**Note: Do not store C-A-T® in two handed application configuration.** Do not thread band through both buckle loops as this will increase difficulty of one-handed/self-application.