

EZ-IO® G3 POWER DRIVER NEEDLE SETS

For use with the EZ-IO® G3 Power Driver


REV112818

Immediate Vascular Access to the central circulation within seconds...

EZ-IO® provides the medical professional immediate vascular access to the central circulation within seconds, delivering medications, intravenous fluids and blood products to adult and pediatric patients. With a specially designed cutting IO needle, manual driver, or small power driver, the EZ-IO® allows the clinician complete control — avoiding the use of force. EZ-IO® provides rapid, smooth entry into the bone's medullary cavity, creating an immediate conduit to the central circulation.

The EZ-IO® Intraosseous Infusion System is the leading choice for intraosseous access across multiple health care settings, and is used by 90 percent of US advanced life support ambulances, over half of US Emergency Departments, and the US Military.

Unlike older IO technology, the EZ-IO®'s patented needle tip has four asymmetrical bevels on the stylet, and two cutting tips on the catheter, providing a smooth and accurate insertion. Black indicator lines are located at 5 mm increments from the hub for assessing insertion depth. With EZ-IO®, insertion depth is completely controlled by the clinician.

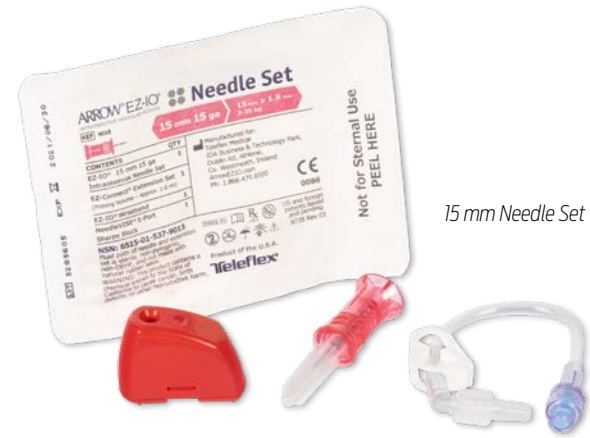

15 mm Needle Set



45 mm Needle Set

EZ-IO Products are restricted to Federal Purchases only. Please contact Customer Service for details.

**Caution: Federal (USA) law restricts this device to sale by or on the order of a practitioner licensed to use or order use of the device or Federal (USA) law prohibits dispensing without a prescription.*



25 mm Needle Set




NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277

PRODUCTS
 WITH A
MISSION®

EZ-IO® G3 POWER DRIVER NEEDLE SETS

REV112818

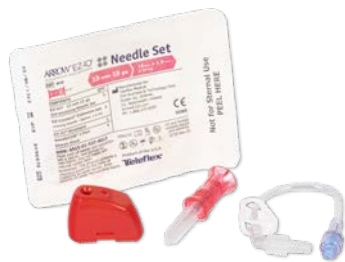

EZ-IO® Power Driver Needle Set Pink Specifications:

- 15 mm 15 Gauge Needle Set
- Designed for use in patients weighing 3-39 kg (approximately 7 - 86 lb)
- For patients with minimal tissue over insertion sites



EZ-IO® Power Driver Needle Set Blue Specifications:

- 25 mm 15 Gauge Needle Set
- Designed for any patient weighing more than 39 kg (over 86 lb)
- For patients who have too much tissue over the insertion site for the 15 mm Needle Set to be used



EZ-IO® Power Driver Needle Set Yellow Specifications:

- 45 mm 15 Gauge Needle Set
- Designed for patients weighing more than 39 kg (over 86 lb)
- For patients with excessive tissue over the targeted insertion site (e.g. edema, large musculature, obesity)
- Ideal for the humerus site in patients over 39kg (over 86 lb)



Features:

- 15 gauge
- Three needle sizes accommodate intraosseous access in patients 3 kilograms and larger
- 304-stainless steel
- Designed for use with any EZ-IO® Power Driver
- NeedleVise® sharps block
- IO wristband (pink for extremity alert)
- EZ-Connect® (IV extension set with a standard Luer connector)
- Black indicator lines are located at 5 mm increments from the hub for assessing insertion depth
- Used by 90 percent of US advanced life support ambulances
- Smooth and accurate insertion

Specifications:

Please see information located next to individual products. (left)

EZ-IO® Power Driver Needle Sets

Color	Size	Item#	NSN#
Pink	15 mm/PD	30-0081	
Blue	25 mm/AD	30-0083	
Yellow	45 mm/LD	30-0085	



STERILE

The EZ-IO® Intraosseous Infusion System is the leading choice for intraosseous access across multiple health care settings.



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

PRODUCTS
WITH A
MISSION®