

# CAT® Carrier

## C3

Concealed  
C-A-T® Carrier



- 1** Make sure the CAT® Tourniquet is in the one-handed configuration.

- 2** Open the C3 Concealed CAT® Carrier as shown to the left. (ensure the product label is visible.)

- 3** Place the CAT® Tourniquet directly over product label with Windlass clip facing flap "C", making sure windlass bar is facing away from product label.

- 4** Fold "A" Flaps over the CAT® Tourniquet

- 5** Fold "B" Flap over and adhere to "A" Flaps already covering the CAT® Tourniquet

- 6** Pull "C" Flap out and up enough to fold over the CAT® Tourniquet and adhere to "B" Flap, making sure the elastic strap fits into the opening of the Windlass Clip.



**NORTH AMERICAN RESCUE®**  
www.NARescue.com • 888.689.6277

**PRODUCTS**  
WITH A **MISSION®**