

CAT[®] Carrier

C3

Concealed
C-A-T[®] Carrier




- 1** Make sure the CAT[®] Tourniquet is in the one-handed configuration.

- 2** Open the C3 Concealed CAT[®] Carrier as shown to the left. (ensure the product label is visible.)


- 3** Place the CAT[®] Tourniquet directly over product label with Windlass clip facing flap "C", making sure windlass bar is facing away from product label.



- 4** Fold "A" Flaps over the CAT[®] Tourniquet

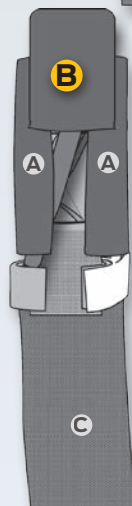

- 5** Fold "B" Flap over and adhere to "A" Flaps already covering the CAT[®] Tourniquet

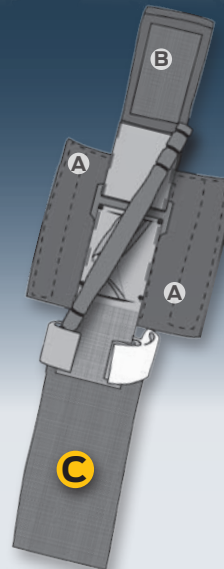

- 6** Pull "C" Flap out and up enough to fold over the CAT[®] Tourniquet and adhere to "B" Flap, making sure the elastic strap fits into the opening of the Windlass Clip.



NORTH AMERICAN RESCUE[®]
www.NARescue.com • 888.689.6277

PRODUCTS WITH A
MISSION[®]