SAM[®] IO Adaptor | Instructions for Use

DESCRIPTION:

The SAM IO Adaptor attaches to the SAM IO Driver to accommodate alternative IO needle designs. The Adaptor snaps into place around the hub of the needle and is then connected to the SAM IO Driver in the same fashion as the SAM IO Needle. Consult all relevant Instructions for Use. Additional product information can be found at sammedical.com

PRODUCT INFORMATION:

Adaptor Ref. number: IO742-EN
Applied Parts: SAM IO Adaptor


IMPORTANT INFORMATION FOR USERS:

For SAM IO Intraosseous Access System products to perform properly, the following conditions are recommended.

- Use this product only in accordance with this manual and applicable product labeling.
- Adjustments, modifications, technical maintenance or repairs are not allowed.
- The SAM IO Adaptor is a single-use device.
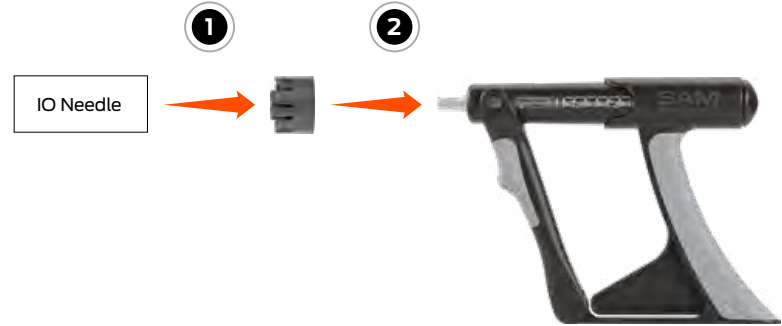
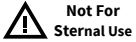
INSTRUCTIONS FOR USE:

1. Attach intraosseous needle to the Adaptor. The Adaptor snaps into place around the hub of the needle.
1. Attach Adaptor to the SAM IO Driver.
2. Note: The SAM IO Needle Assembly is not intended for use with the Adaptor. The SAM IO Needle Assembly should be attached directly to the driver without the Adaptor.
3. Reference the relevant Instructions for Use to complete procedure.

SYMBOL GLOSSARY:

| | |
|---|---|
| For prescription use only. |  |
| Indicates the need for the user to consult the instructions for use. |  |
| Indicates the medical device manufacturer, as defined in EU Directive 93/42/EEC. |  |
| Indicates the manufacturer's batch code so that the batch or lot can be identified. |  |
| Indicates the manufacturer's catalog number so that the medical device can be identified. |  |

| | |
|---|---|
| Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself. |  |
| Not for Sternal use. |  |
| Indicates a medical device that has not been subjected to a sterilization process. |  |
| Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure. |  |

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