

LITTER STANDS

REF 60-0014 • NSN:6530-01-523-3314

REF 60-0029 • NSN:PENDING

PRODUCTS WITH A MISSION[®]




INSTRUCTIONS FOR USE

STEP 1:
To open, unlatch Velcro[®] strap that holds bottom rails of litter stand together during transport.



STEP 2:
Pull legs apart and ensure the crossbar supports locks into place.

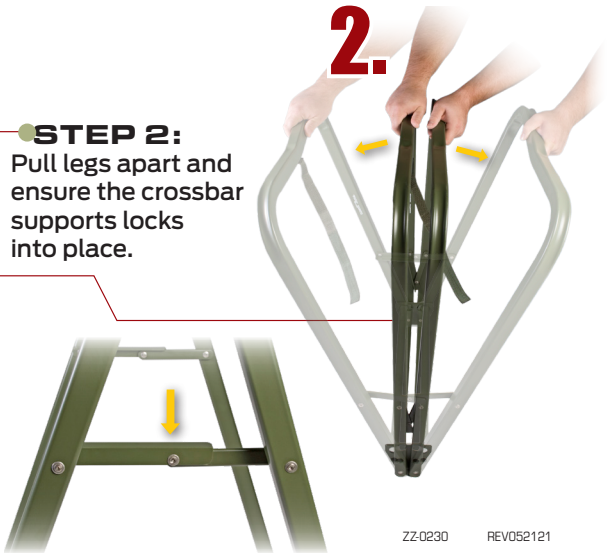

ZZ-0230

REV052121



NORTH AMERICAN RESCUE[®]

MD

www.NARescue.com • 888.689.6277

35 Tedwall Court Greer SC 29650 USA

LITTER STANDS

(INSTRUCTIONS FOR USE CONTINUED)

PRODUCTS WITH A **MISSION**[®]

STEP 3:
Turn litter stand over
and place bottom
rails on floor.



STEP 4:
When positioning the
litter stands, ensure
that the stands are
separated enough to
allow the feet of the
litter to set in between
the two stands.



NOTE:
To close litter stand
repeat steps 1-3 in
reverse order.

NOTE: It is recommended that all litter
stanchions should be weighted to provide
added stability. (Figure 1)



NORTH AMERICAN RESCUE[®]

www.NARescue.com • 888.689.6277

35 Tedwall Court Greer SC 29650 USA