NAR Tactical Suction Device QTY: 1

NAR Taktisk sug ANTAL:1

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MDI Europa GmbH
Langenhagener Str. 71
30855 Hannover-Langenhagen
Germany

Description:**English**

The **NAR Tactical Suction Device** consists of a rigid plastic suction tube, a suction bulb, and a collection bag for aspirated liquids. One way valves prevent the backflow of contents. Because it is manually operated, it requires little maintenance and is not dependent on batteries or other external power sources. It can be easily carried and stored, and it can be instantly made ready for use.

NAR expressly disclaims all implied warranties relating to the product beyond the product's specified purpose; or where used by any party who is not trained or legally authorized to use such product. Use only as directed by your EMS authority or under the supervision of a physician.

Intended Use:

The **NAR Tactical Suction Device** is a small, manual suction device for clearing vomitus and other foreign substances from the upper airway.

Warning:

- The NAR Tactical Suction Device should be used only by persons who have received training on suctioning the oral airway. Improper use could result in injury to the patient. Use only as directed by your EMS authority or under the supervision of a physician.
- Within the European Union (EU) the Tactical Suction Device is not indicated for pediatrics or pregnant women due to EU regulatory requirements.
- Use of this device may result in your contact with contaminated body fluids. Use appropriate body substance isolation measures along with personal protective equipment such as gloves and eye protection.
- In the event of a Tactical Suction Device malfunction follow local protocols and report any serious incident to North American Rescue or authorized representative and the competent authority of the Member State.

 **Instructions for Use**

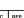
- Remove the NAR Tactical Suction Device from the package. Pull the suction tube out until it stops at the red indicator.
- Clear any obvious foreign substances from the patient's airway by using finger sweeps.
- Grasp the suction bulb between your thumb and forefingers.
- Squeeze the suction bulb.
- Insert the suction tube into the back of the patient's throat. Do NOT insert farther than you can see.
- Release the suction bulb to remove the liquid from the patient's throat.
- Continue to squeeze and release the bulb to remove additional liquid.
- After the airway is cleared, it may be necessary to take other steps to clear and maintain the airway. Provide oxygen or assist ventilations as indicated in local protocols.

Caution:

Inserting the suction tube beyond visibility may restrict patient airway. Overfilling of the collection bag will result in potentially infectious fluid being released from the air vent.

Disposal:

The NAR Tactical Suction Device is a single use device. It is designed for disposal after use. Do not attempt to clean or reuse the device, as this may increase the possibility of cross contamination. Dispose of in a manner ensuring the isolation of potential infectious substances in accordance with universal protocols.

- | | | |
|--|---|---|
|  WARNING:
Cancer
-www.
P65Warnings.ca.gov |  LOT Lot number |  Consult instructions for use |
| |  M Manufacturer |  MD Medical Device |
| |  Do not use if package open or damaged |  TM Country of Manufacture |
| |  S Single use |  cc rep Authorized Representative |
| |  REF Device Part Number |  ☀ Keep Away From sunlight |
| |  ☔ Keep Dry |  ⌚ Use-by Date |

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