

BLEEDING INTERVENTION TRAINER - GSW

The Gunshot Wound (GSW) Bleeding Intervention Trainer is a mobile solution specifically designed for point-of-injury trauma simulation and hemorrhage control. The system incorporates a versatile fluid delivery mechanism, featuring a fluid bag that supports both gravity-fed and manual pump operation, allowing precise control over the delivery of simulated blood.

The Gunshot Wound (GSW) in a Box® task trainer simulates a gunshot to soft-tissue, used to instruct students in proper wound packing technique. The representative skin and tissue of this product is constructed of high quality silicone rubber, with layers of coloration that provide a depth and realism to the product. The PATENTED construction with SOFTTECH® materials a realistic feel, and is very durable and UV resistant. Blood tubing is integrated into the construction to easily create simulated blood flow. The task trainer is contained in its own storage case.

Construction

The representative skin and tissue of the Gunshot Wound (GSW) in a Box® Simulator is constructed of high-quality silicone rubber, with layers of coloration that provide a depth and realism to the product. In addition, the SOFTTECH® material has a realistic feel, and is very durable and UV resistant.

Operation

The Gunshot Wound (GSW) in a Box® is contained in its own storage case. The task trainer reinforces wound packing technique for this unique mechanism of injury. Built within a durable Pelican 1500 case, this durable trainer offers two stands for fluid reservoirs to simulate massive hemorrhage.

Maintenance

The device should be rinsed in clean, warm water and allowed to dry before storage. The simulation may be removed from the storage case for cleaning. The device may be washed with mild detergents like dish soap. Allow the device to dry fully before storage to prevent mold or mildew growth.



Learn more about this and
other Products with a Mission®
at www.NARescue.com



FEATURES/SPECIFICATIONS

- Easily transportable, yet rugged enough for classroom and field use
- Set up/ Break down in less than a minute

- Highly Adaptive Bleeding Control
- Focused Hemorrhage Control Training
- Patented materials give this product a realistic feel, while still being durable and UV resistant

BLEEDING INTERVENTION TRAINER - LACERATION

ITEM #

93-1117

DIMENSIONS


- Measurements (Exterior): L 18.5 in. x W 14.06 in. x D 6.93 in. (47 cm. x 35.7 cm. x 17.6 cm.)
- Weight: 14 lbs.



NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277

BLEEDING INTERVENTION TRAINER - GSW

all KITS INCLUDE:

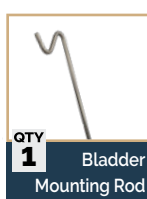

Learn more about this and other Products with a Mission® at www.NARescue.com



KIT CONTENTS

- 1 x Wound in a Box - Gun Shot Wound (GSW)
- 1 x Blood Supply Pump with Carry Bag
- 1 x Bladder Mounting Rod
- 2 x Simulated Blood - 8 oz.
- 8 x NAR® Wound Packing Gauze
- 2 x QuikClot® Combat Gauze Trainer
- 1 x Portable Bleed Tray
- 1 x Pelican Hard Case

BLEEDING INTERVENTION TRAINER - LACERATION

ITEM #

93-1117



NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277