

MILITARY GRADE FIELD SHIELD®

WOUND DRESSING

First Of Its Kind, Spray-On Dressing

The Military Grade Field Shield® Wound Dressing is a spray on hydrogel wound dressing that hydrates, seals and protects dermal injuries to create and maintain a moist wound environment known to be conducive to wound healing. Field Shield® is a FIRST of its kind, spray-on dressing that contains both antimicrobial silver hydrosol and lidocaine for rapid pain relief when applied. This unique combination wound dressing is the first to receive FDA 510k clearance, hitting all the main needs to ensure effective and complete wound healing.

With active lidocaine, the spray on dressing delivers anesthetic lidocaine that can relieve pain within 30 minutes of application to the wound or skin. Field Shield® also contains a powerful silver hydrosol that has been shown to kill bacteria and fungi within the dressing. The silver hydrosol may inhibit the growth of microorganisms such as Staphylococcus aureus, Pseudomonas aeruginosa, Escherichia coli, antibiotic resistant strains of MRSA and VRE, as well as fungi such as Candida albicans and Candida auris within the dressing.

It is a water-based hydrogel system that donates moisture to the wound and helps maintain a moist wound environment. The dressing then sets into an elastic, semi-occlusive, transparent film barrier over the surface of the wound within 5 minutes of application. The film sets over intact and compromised skin surfaces and is intended for use for up to 30 days, with reapplication recommended every 24 to 72 hours.

FEATURES/SPECIFICATIONS

- Soothing hydrogel spray with lidocaine, reduces pain within minutes
- Silver hydrosol KILLS multi drug-resistant bacteria & fungi in less than 10 minutes
- Mitigates burn progression/expansion when applied early in field care
- Elastic nanopolymer barrier seamlessly covers any size and shape wound and blocks dirt and bacteria
- Promotes early healing with ideal moisture balance
- Military grade, Rx only version of the Field Shield® Wound Dressing Spray
- +90 sprays for .05 oz
- +150 sprays for 1 oz.

Learn more about this and other Products with a Mission® at www.NARescue.com



MILITARY GRADE FIELD SHIELD®

ITEM #	VOL	NSN
75-0032	0.5 oz.	----
75-0037	0.5 oz. (24 Pk)	----
75-0034	1 oz.	----
75-0039	1 oz. (15 Pk)	6510-01-727-1966

* Explore other Field Shield® and KeriCleans® products available at www.NARescue.com

DIMENSIONS

0.5 fl oz.

- Package: H 4 in. x W 0.75 in.

1 fl oz.

- Package: H 4.25 in. x W 1 in.

R_X ONLY



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

MILITARY GRADE FIELD SHIELD®

WOUND DRESSING

FIELD SHIELD® WOUND APPLICATION GUIDE

1 Shake Field Shield bottle well before use. Clean & disinfect area if necessary.




2 Press pump firmly to apply mist, repeating until area is fully covered.



3 Dab around area to remove excess solution. DO NOT RUB IN.



4 Protection begins once the solution has set into thin film (typically within 1-2 minutes).

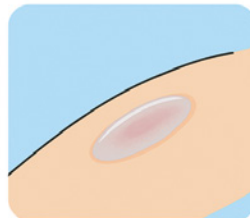

5 Wipe or scrape nozzle surface prior to and after each use to remove film & prevent clogging.



6 Reapply every 24-72 hours by following steps 1-4. Remove if necessary by cleaning or debriding.



OPTIONAL:
Use a secondary dressing if necessary for exudate management or for additional protection of the wound, especially in high impact areas. Secondary dressing can be applied once Field Shield has set.

FIELD SHIELD® IN ACTION

Below are a few examples from KeriCure® Medical showing the Field Shield Wound Dressing applied to different types of wounds.

MARINE FACIAL LACERATION



Day 1 Post Injury



Day 3 with Treatment Applied



Fully Healed

MOTORCYCLE ACCIDENT




Initial Injury



1 Day Post Treatment



1 Week Post Treatment

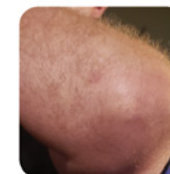

Fully Healed after 2 Weeks

SHIN PUNCTURE WOUND



Initial Injury



Post Treatment



Fully Healed

Learn more about this and
other **Products with a Mission®**
at www.NARescue.com



NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277