



NORTH AMERICAN RESCUE®


EZ-Tourniquet Pouch Gen 2

INSTRUCTIONS FOR USE


Prepare the C-A-T® Tourniquet.



Ensure windlass retention strap is not over the clip. Ensure windlass is not locked in the windlass retention clip.

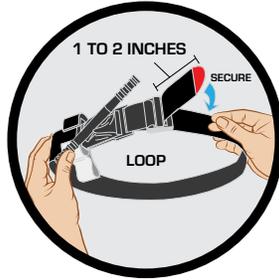

Adjust the self-adhering band until only 1-2 inches have been passed through the single routing buckle. Adhere the 1-2 inch of the band to itself.

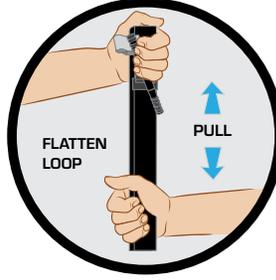


Flatten the loop formed by the self-adhering band. Hold the hardware of the C-A-T® Tourniquet in one hand, and place your other hand at the opposite end of the flattened loop.




Fold the band into 3 equal lengths in a "Z-fold" pattern. Flatten the folded sections together. Hold them parallel so the C-A-T® Tourniquet collapses into a "N" pattern. (In 3 equal lengths)


Prepare the EZ-Tourniquet Pouch Gen 2

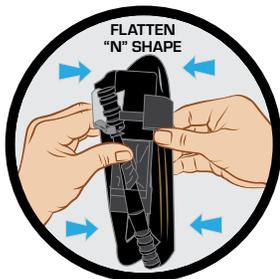


Windless clip and Single routing buckle will be on one side and self-adhering band on the other side. Flatten.

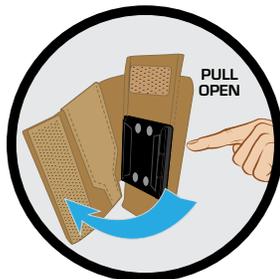

Release the hook and loop closure on the front of the pouch and open to expose the rail system completely.

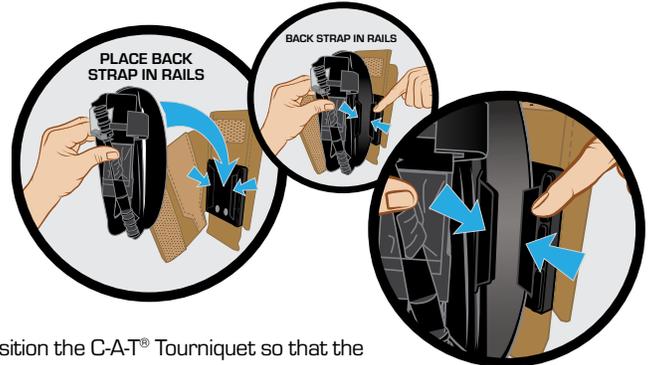

Position the C-A-T® Tourniquet so that the windlass is facing the front of the pouch, and insert the self-adhering band into the plastic rail.

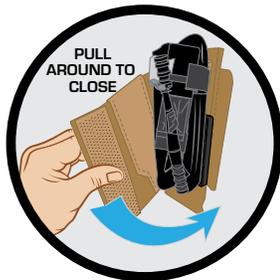

Position the C-A-T® Tourniquet within the pouch and close the pouch by securing the hook and loop closure.



Push the C-A-T® Tourniquet to the bottom of the pouch to secure it.



Attach the pouch to vest or gear using the MOLLE system.

Instructions Continued on Reverse



NORTH AMERICAN RESCUE


EZ-Tourniquet Pouch Gen 2

INSTRUCTIONS FOR USE

INSTRUCTIONS FOR USE (cont'd)

How to Use

The C-A-T® is now ready for single handed deployment. Simply remove the C-A-T® Tourniquet from the holder with a decisive pull and it will be deployed in a large open loop for easy self application.

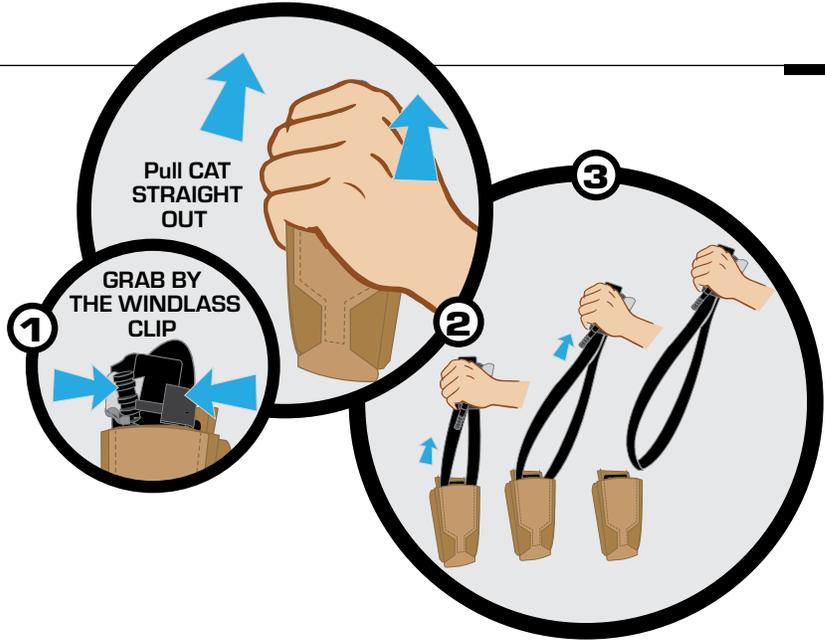

Color options and Item Numbers



REV. 022426

EZ-Tourniquet Pouch Gen 2 Instructions