

Adjustable IV Pole

REF 60-0008-EU

NSN: 6530-01-504-9054

ZZ-2021-EU • REV030926

Instructions For Use

Intended Use: The North American Rescue® Adjustable IV Pole is a stationary or movable stand intended to hold infusion liquids.

Product Description: The IV Pole is designed to be attached to all NAR® framed litter platforms.

Indications For Use: The IV Pole is indicated to support and position intravenous (IV) bags and associated medical accessories during patient care.

Contraindications: None

User Groups: The IV Pole is intended to be used by emergency responders and medical professionals including military medics and Emergency Medical Services (EMS) professionals.

Instructions For Use:

- 1.** The NAR® IV pole is designed to fit pre-molded holes of all NAR® framed litter platforms.
- 2.** The IV Pole is fully adjustable from 18.5" retracted to 32" fully extended. The middle locking nut twists to the right to allow for deployment and left to lock.
- 3.** IV bags or other infusion bags can be hung from the IV pole once in place and locked.

Warnings and Precautions: Should not come into direct contact with the casualty's skin.



MDI Europa GmbH
Langenhagener Str. 71
30855 Hannover-Langenhagen
Germany



In the event of a malfunction, follow local protocols and report any serious incident to North American Rescue® or authorized representative and the competent authority of the member state.



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA