

T3 LIGHTWEIGHT ASSAULT LITTER

REF 60-0022 • NSN: 6530-01-663-1222



INSTRUCTIONS FOR USE

PRODUCTS WITH A MISSION[®]

- **STEP 1:**
While standing upright, release buckles from litter.



- **STEP 2:**
Place litter on the ground and completely extend with the fabric side up.



NOTE:
When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.



Figure 1

Relaxed
Position

ZZ-2022 • REV031226



NORTH AMERICAN RESCUE[®]

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

T3 LIGHTWEIGHT ASSAULT LITTER

INSTRUCTIONS FOR USE (CONT.)

- **STEP 3:** Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

NOTE:
This action is best executed simultaneously using 2 individuals on each end of the litter.

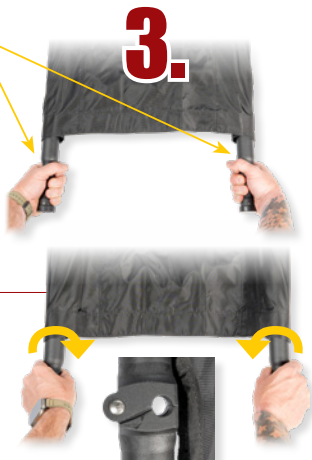


Figure 2

- **STEP 4:** While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.

NOTE:
Apply pressure to spreader bars in a straight line parallel to the bed and poles for best results.



NOTE: To close and store litter, repeat the directions in reverse order.

PRODUCTS WITH A MISSION[®]



NORTH AMERICAN RESCUE[®]

www.NARescue.com • 888.689.6277