

RAVEN® LITTER SERIES

Raven® II Model 90C Litter

REF 60-0001 • NSN: 6530-01-432-5114



INSTRUCTIONS FOR USE

PRODUCTS WITH A MISSION®

STEP 1:

While standing upright, release buckles from litter.



STEP 2:

Place litter on the ground and completely extend with the fabric side up.

NOTE:
When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.

Relaxed Position



Figure 1

ZZ-0227 • REV030326



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court Greer, SC 29650 USA

RAVEN® LITTER SERIES

INSTRUCTIONS FOR USE (CONT.)


PRODUCTS WITH A **MISSION®**

● **STEP 3:**

Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

NOTE:

This action is best executed simultaneously using 2 individuals on each end of the litter.

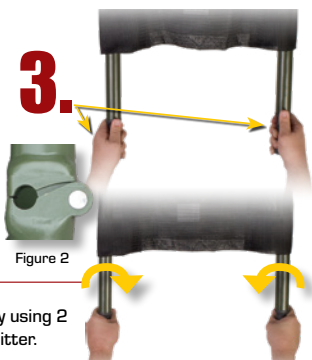

● **STEP 4:**

While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.

NOTE:

To close and store litter, repeat the directions in reverse order.



● **STEP 5:**

Place patient on litter, securing the straps around the patient, buckle and pull straps tight to avoid patient movement during transport.



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277