

NAR® Litter Backrests

Talon®/Raven® Litter Backrest: [REF] 60-0027 • NSN: 6530-01-594-7444

OSL Litter Backrest: [REF] 60-0024



INSTRUCTIONS FOR USE

PRODUCTS WITH A **MISSION®**

- **STEP 1:**
Unsnap the backrest fabric straps and loosen backrest fabric.

NOTE:
This allows for the backrest frame to spread and fit over the sides of the litter.



- **STEP 2:**
Spread backrest frame and attach mounting brackets to litter poles on each side of the litter.

NOTE:
Attachment point to the litter poles should be inside and against the litter feet.

NOTE: For 45° configuration, place backrest on litter as shown above. For 90° configuration flip backrest over and place on litter.

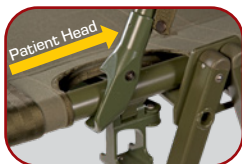


Figure 1: 45° Configuration

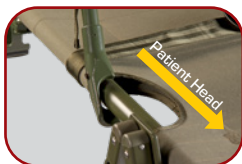


Figure 2: 90° Setup Configuration
(flip backrest over)

ZZ-0292 • REV030926



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

NAR[®] Litter Backrests

Talon[®]/Raven[®] Litter Backrest: [REF] 60-0027 • NSN: 6530-01-594-7444

OSL Litter Backrest: [REF] 60-0024

PRODUCTS WITH A MISSION[®]

● STEP 3:

Tighten backrest fabric by snapping the backrest fabric straps on each side.



45° Configuration

● STEP 4:

To remove the backrest, unsnap the backrest fabric straps, loosen the backrest fabric and remove the backrest frame from the litter poles.



90° Configuration

NOTE: The backrest is made to fit snugly onto the OSL™ Litter. Therefore, it may take some effort to completely install the backrest.



Backrest in 45° Configuration



Backrest in 90° Configuration



NORTH AMERICAN RESCUE[®]

www.NARescue.com • 888.689.6277