

LITTER STANDS

REF 60-0014 • NSN:6530-01-523-3314

REF 60-0029 • NSN:PENDING



INSTRUCTIONS FOR USE

PRODUCTS WITH A **MISSION**[®]


- **STEP 1:**
To open, unlatch Velcro[®] strap that holds bottom rails of litter stand together during transport.

1.



- **STEP 2:**
Pull legs apart and ensure the crossbar supports locks into place.

2.

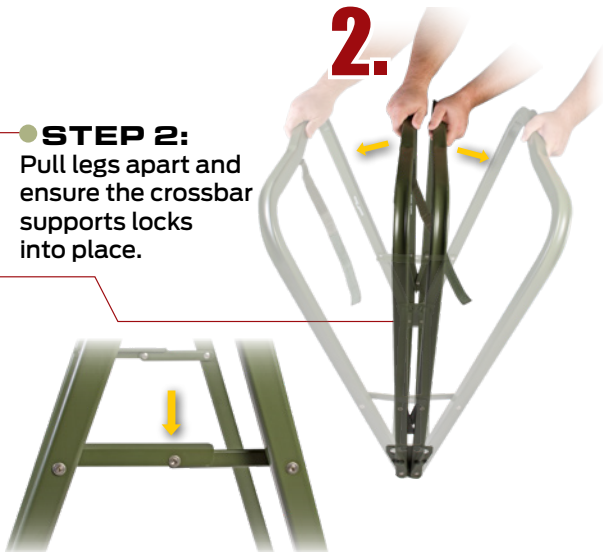

ZZ-0230 • REV030526



NORTH AMERICAN RESCUE[®]


www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

LITTER STANDS

INSTRUCTIONS FOR USE (CONT.)

STEP 3:
Turn litter stand over
and place bottom
rails on floor.

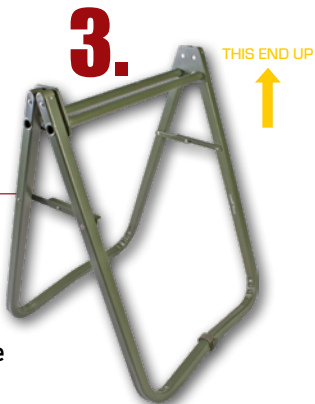


STEP 4:
When positioning the
litter stands, ensure
that the stands are
separated enough to
allow the feet of the
litter to set in between
the two stands.

NOTE:
To close litter stand,
repeat
steps 1-3 in
reverse order.



NOTE: It is
recommended that all
litter stanchions should
be weighted to provide
added stability.
(Figure 1)

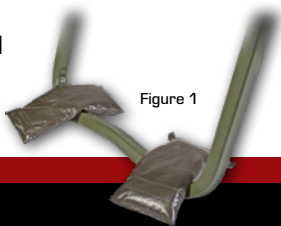

NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA