

NAR® LITTER STANDS



MAINTENANCE INSPECTION

Although all NAR® litter stands are designed for use in rugged conditions, they still may require regular preventative maintenance.

INTENDED USE

The NAR® Litter Stands are stationary supports intended to immobilize casualty extraction devices.

PRODUCT DESCRIPTION

The NAR® Litter Stands are designed to provide a stationary support system for compatible litter platforms.

INDICATIONS FOR USE

The NAR® Litter Stands are indicated for use as stationary supports intended to immobilize compatible litter platforms in field environments.

CONTRAINDICATIONS

None

USER GROUPS

The NAR® Litter Stands are intended to be used by emergency responders and medical professionals including military medics and Emergency Medical Services (EMS) professionals.

PRODUCTS WITH A MISSION®



ZZ-0306 • REV030626



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer, SC 29650 • USA

NAR[®] LITTER



SAFETY INSPECTION

PRODUCTS WITH A **MISSION[®]**

To maintain service life and high performance, all belts, hinges, handles, and other materials must be inspected frequently. Periodic tests are to be conducted by a trained inspector. Inspect your equipment daily, and replace it if any defective conditions are found.

- Inspect frame, bars, and welds points for cracks, distortions, or stress marks.
- Check pins to ensure they are in place and have fasteners attached properly.
- Inspect hinges for cracks, distortions, or stress marks.



NORTH AMERICAN RESCUE[®]

www.NARescue.com • 888.689.6277