

LifeFlow PLUS Blood & Fluid INFUSER from 410 Medical

REV040326

LifeFlow® PLUS is a hand-operated rapid infusion device that allows health care providers to deliver blood and fluids quickly and effectively, improving resuscitation of critically ill or injured patients. Delivering in 10mL increments with each handle compression, a LifeFlow PLUS user can administer 1 unit of blood or 500ml of fluid in less than 2 minutes, maintaining full control over the volume delivered.

The syringe size allows the user to overcome the resistance of IV catheters and automatically refills when the handle is released - allowing the user full control of volume of fluid delivered. This also protects the syringe from bacterial contamination.

LifeFlow PLUS can be operated with one hand, leaving the operator's other hand free to address other clinical issues, monitor the infusion site, and assess patient response. This is particularly useful in austere environments and transport when resources are limited, and personnel are performing multiple tasks simultaneously.

Current methods of blood and fluid delivery are too slow to quickly restore intravascular volume in patients with severe shock and hypotension. For these patients "time is tissue," meaning every minute of continued hypotension can result in more organ injury. LifeFlow PLUS offers a simple way to restore perfusion with rapid and controlled delivery of fluids and blood.

Features:

- Hand-operated, rapid infusion device that allows health care providers to deliver fluid & blood products quickly and effectively
- Deliver 500ml of fluid in less than 2 minutes through a 20G IV catheter
- Deliver 1 unit of whole blood in 2 minutes
- Goes from the field to the hospital
- Can be operated with one hand
- Handle and tubing packaged together for quick and easy set-up
- Designed to protect the syringe from bacterial contamination



Rx ONLY

LifeFlow PLUS BLOOD & FLUID INFUSER

ITEM #

30-0287

Specifications:

- 12 in. x 7.5 in. x 2.75 in.
- Weight: 0.95 lb

LifeFlow[®]
VOLUME RESUSCITATION. WHEN MINUTES MATTER.



NORTH AMERICAN RESCUE[®]

www.NARescue.com • 888.689.6277


PRODUCTS

WITH

MISSION[®]

LifeFlow PLUS Blood & Fluid INFUSER

REV040326

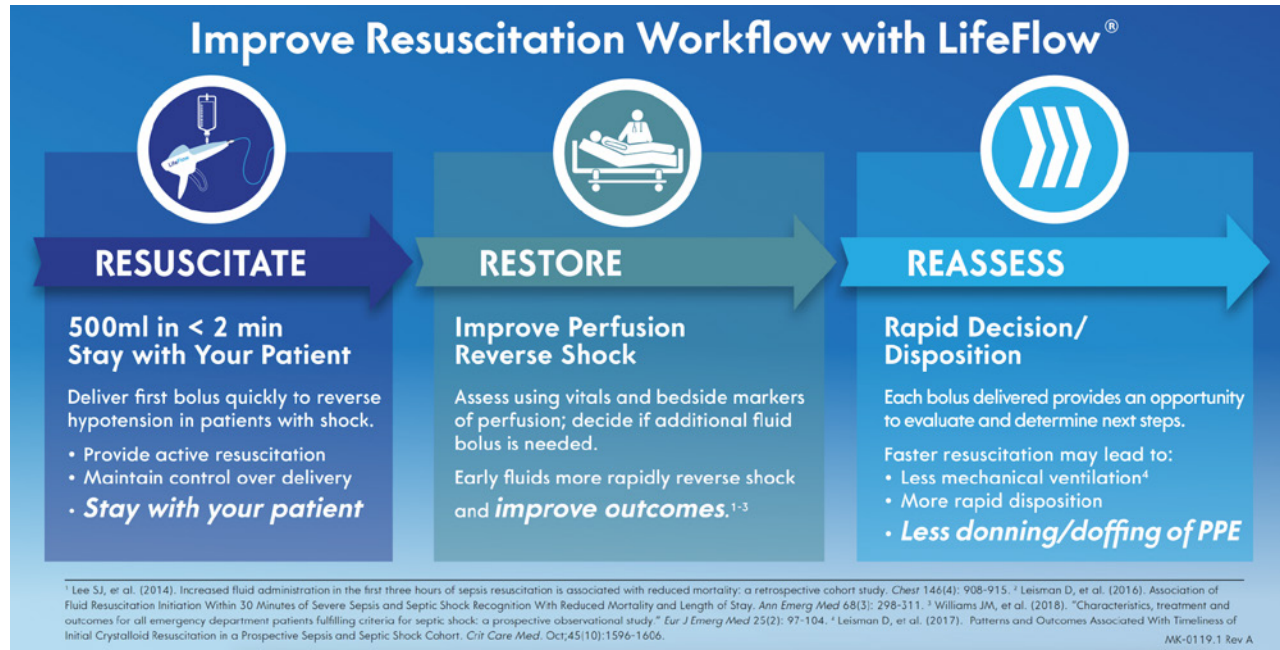

This product requires Medical Device Authorization for purchase.

SPEED Average time to set up and prime LifeFlow PLUS is less than 3 minutes. LifeFlow PLUS can deliver 1 unit of whole blood or 500 ml of fluid in less than 2 minutes through a 20G IV catheter.

CONTROL LifeFlow PLUS delivers fluid in 10mL increments with each handle compression. This syringe size allows the user to overcome the resistance of IV catheters. The syringe automatically refills with handle release, allowing the user full control of volume of fluid delivered.

EFFICIENCY LifeFlow PLUS can be operated with one hand, leaving the operator's other hand free to address other clinical issues, monitor the infusion site, and assess patient response. This is particularly useful in pre-hospital, emergency department, and ICU settings where speed and efficiency are important and personnel resources may be limited.



Air Check™
makes priming and setup easier and will stop the flow of fluid if air enters the system

Clear Canopy
Syringe is protected from contact & bacterial contamination during use

Fast and Accurate
10ml of fluid with each pull

Force Reducer™
smooths the pulsatile flow, reducing the amount of effort required by the user



NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277