

AAJT-S APPLICATION & REMOVAL GUIDE

STEP-BY-STEP INSTRUCTIONS FOR SAFE AND EFFECTIVE USE

WWW.COMPRESSIONWORKS.COM



compressionworks


REMOVAL

1



DEFLATE THE
BLADDER SLOWLY.

2

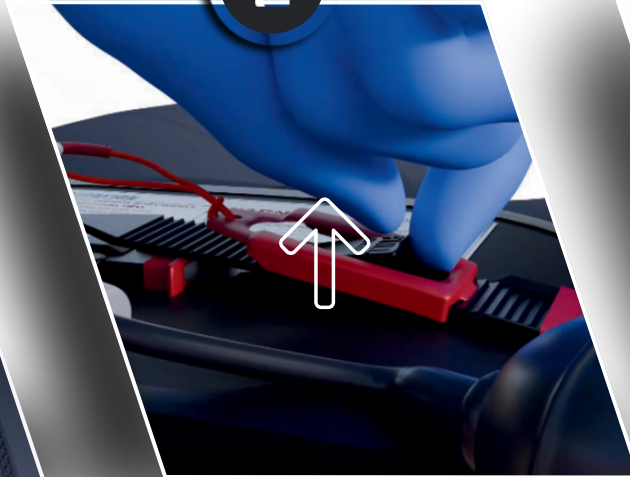

LIFT AND RELEASE
THE BLACK PULL TAB.

3



REMOVE THE
LADDER STRAP.

APPLICATION

1



CONNECT
RED TO RED.

2



POSITION
THE DEVICE

3



REMOVE SLACK
FROM BELT

4




RATCHET TO
TIGHTEN

5



INFLATE BLADDER
UNTIL **GREEN**

DO NOT CUT

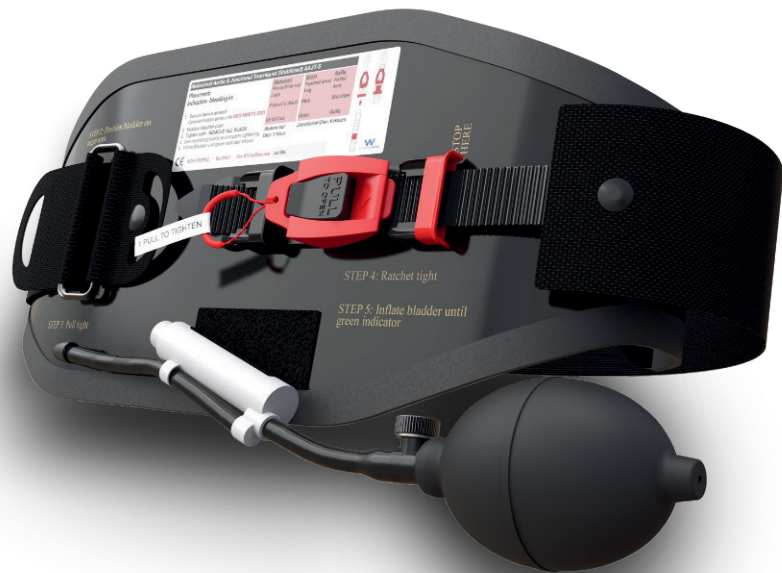

PREMATURE REMOVAL
MAY RESULT IN
RESUMED BLEEDING



SCAN THE QR CODE
TO WATCH A
HOW-TO-USE VIDEO.