



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

NAR Lightweight Wheeled Litter Carrier Instructions

Version 1.0 • September 2015

Lightweight, adjustable &
stowable for transport



*NOTE: Litter
sold separately.*

PRODUCTS WITH A
MISSION®



NOTE: Litter
sold separately.

NAR Lightweight Wheeled Litter Carrier with Case

Lightest wheeled litter cart on the market for single person transport to & from MedEVAC vehicles, aircraft, etc.

- Locking clamp system to keep the litter firmly in place, and it allows for different litter widths
- Compatible with Talon® II 90C, Raven® 90C, MedEvac & T3 litters as well as all NATO litters
- Performs as litter stand in the field due to it's unique wheel locking system
- Quick and easy to deploy or disassemble for storage
- Includes rugged, wheeled, stackable hard case with custom laser-cut foam packing interior

Color	Item#	NSN#
ODG	60-0063	N/A

Carrier Dimensions Open:

L 56.29 in. x W 27.5 in. x H 36.2 in.

Carrier Dimensions Half-Folded:

L 49.6 in. x W 27.5 in. x H 19.6 in.

Carrier Dimensions Folded:

L 42.1 in. x W 27.5 in. x H 9.84 in.

Litter Carrier Weight:

38 lb 8 oz (17.5 kg)

Litter Carrier with Case Weight:

88 lb (39.9 kg)

Litter Carrier Payload:

550lb (250 kg)

Case Dimensions:

H 12.5 in. x W 48 in. x D 29.5 in.

Wheeled Litter Carrier ONLY

ITEM #: **60-0041**

NATO NSN# **6530-01-663-2308**
is also available for purchase.



Step 1

Unlock and open the Wheeled Litter Carrier protective case. Remove the Wheeled Litter Carrier components from the case to begin assembly.



Step 2

Insert axle rod of tire into sleeve while pushing down on locking lever. Once axle is fully seated, release locking lever to secure rod in place. Repeat this step for the second tire.





Step 3

Using hand, push the frame stabilization bar forward until it locks into place (tire side).





Step 4

Lift up on the handle side of the frame until the legs are extended, supporting the weight of the litter carrier. Using hand/foot, push the frame stabilization bar forward until it locks into place (handle side).



Step 5

Pull out the handle of litter carrier while rotating 90 degrees counter clockwise. Handle will lock itself into place. Repeat this step for the second handle.





*NOTE: Litter
sold separately.*

Step 6

Place litter onto carrier, ensuring that the litter poles are properly seated in the provided channels.





*NOTE: Litter
sold separately.*



Step 7

Secure the litter to the carrier by moving the C clamp over the top of the litter poles. Once the clamp is properly placed, push down the locking lever to secure. Repeat this step for all four clamps.





Step 8

To engage wheel lock, rotate the lock hand counter-clockwise until lock fully engages against the wheels. Repeat this step for the second lock. When ready to transport, release wheel locks by rotating the lock hand clockwise.

Also Available...

Talon® II 90C Collapsible Handle Litter

*Advanced, innovative combat evacuation system
featuring portable quad-fold design & retractable handles*

- Standard 90 in. NATO-compatible evacuation platform when fully extended
- Developed to U.S. Army requirements. Certified Safe-to-Fly U.S. Army Rotary Wing
- Retractable, ergonomic handles and automatic locking hinges
- Slip-resistant, chemical-resistant rip-stop fabric tested & approved by Federal & private labs using live warfare agents
- Includes 6 IV-attachment points and 2 patient-securing straps



Item#	NSN#
60-0002	6530-01-504-9051

BLK

Litter Dimensions:

Folded w/handles collapsed -
H 20.5 in. x W 8.5 in. x D 9.5 in.

Open w/handles collapsed -
L 78 in. x W 22.5 in. x D 6 in.


Open w/handles extended -
L 90 in. x W 22.5 in. x D 6 in.

Wt: 15 lb 6 oz (6.97 kg)

Payload: 1,200 lb (544.3 kg)

Ground Clearance: 1.5 in.

U.S. Patents: 6,842,923 &
5,598,592, Patent(s) Pending

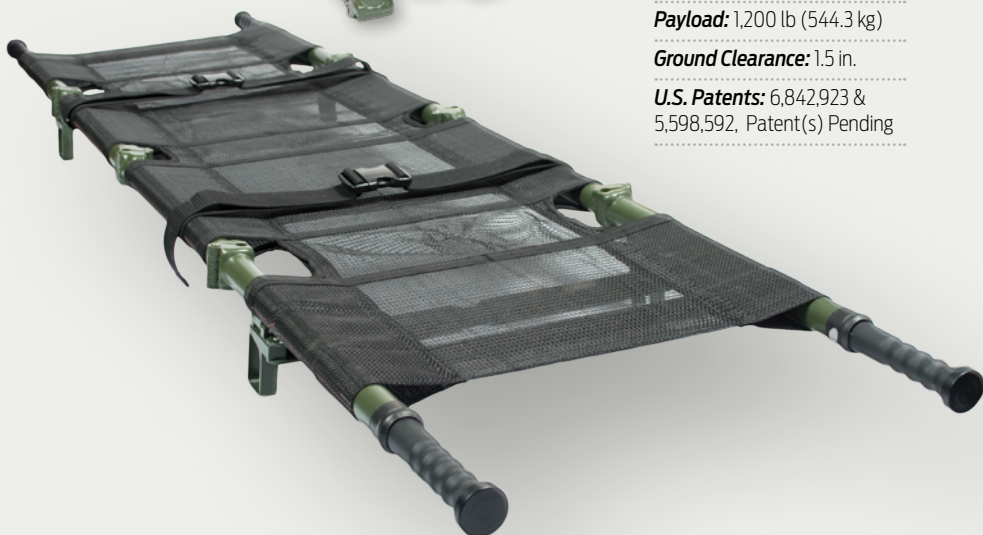



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

**PRODUCTS WITH A
MISSION®**