

# TALON® II LITTER SERIES

Talon® II 90C Collapsible Handle Litter

PN:60-0002 • NSN: 6530-01-504-9051

T3™ Lightweight Assault Litter

PN:60-0022

## INSTRUCTIONS FOR USE

PRODUCTS WITH A MISSION®

### ● **STEP 1:**

While standing upright, release buckles from litter.



### ● **STEP 2:**

Place litter on the ground and completely extend with the fabric side up.

#### NOTE:

When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.



Figure 1



Relaxed Position



**NORTH AMERICAN RESCUE®**

[www.NARescue.com](http://www.NARescue.com) • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

# TALON® II LITTER SERIES

(INSTRUCTIONS FOR USE CONTINUED)


PRODUCTS WITH A MISSION®

## STEP 3:

Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

### NOTE:

This action is best executed simultaneously using 2 individuals on each end of the litter.

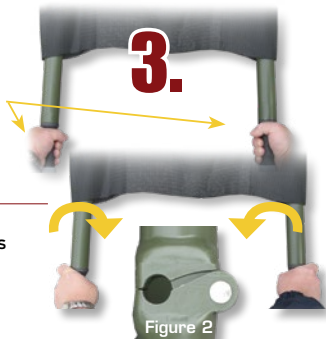

## STEP 4:

While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.

### NOTE:

Apply pressure to spreader bars in a straight line parallel to the bed and poles for best results.



NOTE: To close and store litter, repeat the directions in reverse order.

## Collapsible Handle Option:


Stand upright and extend handles to full length.

### NOTE:

The button will pop out when the handle is securely extended. Depress the button in order to collapse the handles.

### NOTE:

To aid in deployment extend handles to full length before attempting to apply pressure to spreader bars. Ensure extendable handles are fully collapsed before collapsing spreader bars.

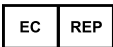

MDI Europa GmbH  
Langenhagener Str. 71  
30855 Hannover-Langenhagen  
Germany

PN:ZZ-0029 REV.030618



**NORTH AMERICAN RESCUE®**

[www.NARescue.com](http://www.NARescue.com) • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA