

RAVEN® LITTER SERIES

Raven® II Model 90C Litter
PN:60-0001 • NSN: 6530-01-432-5114

INSTRUCTIONS FOR USE

PRODUCTS WITH A MISSION®

STEP 1:
While standing upright, release buckles from litter.



STEP 2:
Place litter on the ground and completely extend with the fabric side up.

NOTE:
When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.

Figure 1



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

RAVEN® LITTER SERIES

Raven® II Model 90C Litter
PN:60-0001 • NSN: 6530-01-432-5114


(OPENING INSTRUCTIONS CONTINUED)

PRODUCTS WITH A **MISSION®**

- **STEP 3:** Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

NOTE:


This action is best executed simultaneously using 2 individuals on each end of the litter.

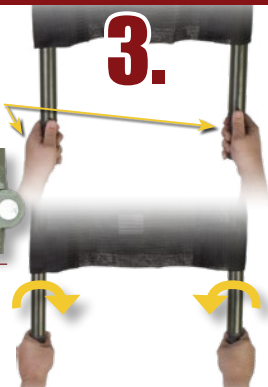



- **STEP 4:** While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.

NOTE:

To close and store litter, repeat the directions in reverse order.

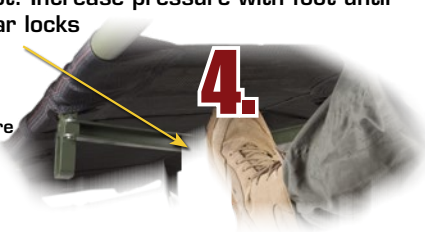

- **STEP 5:** Place patient on litter, securing the straps around the patient, buckle and pull straps tight to avoid patient movement during transport.

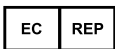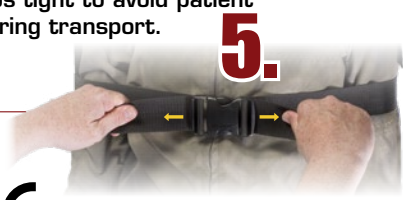

MDI Europa GmbH
Langenhagener Str. 71
30855 Hannover-Langenhagen
Germany



PN:ZZ-0227 REV.042319



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA