

Directions for Use

CAUTION:

Surgical airway training, kit familiarization, and the adherence to established protocol are required for use of this product.

STEPS:

1. **Prepare** equipment (test distal cuff, follow established protocol).
2. **Stabilize** thyroid cartilage and **Locate** cricothyroid membrane.
3. **Cleanse** site.
4. **Vertically incise soft tissue** over cricothyroid membrane (approximately 3-4 cm incision).
5. Confirm anatomy, and **horizontally incise cricothyroid membrane**.

Maintain cricothyroid **opening:**

Option 5a: **Insert tracheal hook** into cricothyroid opening and suspend thyroid cartilage.

Option 5b: **Insert finger** into cricothyroid opening.

6. **Insert Bougie** through cricothyroid opening, and **confirm tactile feedback** from tracheal rings.
7. **Advance Bougie-Tube assembly**, and **inflate cuff** (appropriate volume between 5-10 cc of air).
8. **Remove Bougie Assembly**.
9. **Ventilate patient, reconfirm tracheal placement** and **secure** tube.

Confirmation, and reconfirmation of correct tube placement may include, but is not limited to:

Tracheal ring "clicking" or "tactile feedback" from inserted Bougie; chest rise and fall during ventilation; presence of bilateral breath sounds with auscultation; absence of epigastric sounds with auscultation; evidence of sustained ETCO₂ with ventilation; negative finding of esophageal detection device.



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

BAC-Pack

BOUGIE AIDED CRICOTHYROIDOTOMY PACK

Indications: Patient who cannot be ventilated or intubated by non-surgical method.

Contraindications: Patient who can be ventilated by any means other than placement of surgical airway.

Caution: Federal Law restricts this device to sale by, or on the order of, a licensed physician.

Warranty: The product, and the components contained in its package, constitute a medical device for which specific training is required. North American Rescue, LLC. warrants this product as merchantable and fit for its specific purpose. North American Rescue disclaims all other expressed or implied warranties relating to this product; this includes any use beyond product's specified purpose; or use by any untrained, or legally unauthorized party.

PN: 10-0052

KIT CONTENTS:

- 1X Swabstick, Chloraprep
- 1X Prep Pad, Alcohol
- 1X Gauze, Pad - 4" X 4" (1 pk Of 2)
- 1X #10 Scalpel, Penblade
- 1X Tracheal Hook
- 1X NAR Tracheostomy Kit w/Bougie-Introducer
- 1X 10cc Syringe
- 1X Tracheal Tube Holder

R_X ONLY




Distributed by
NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277


35 Tedwall Ct. • Greer SC 29650 • USA

QUICK REFERENCE GUIDE


1. Prepare

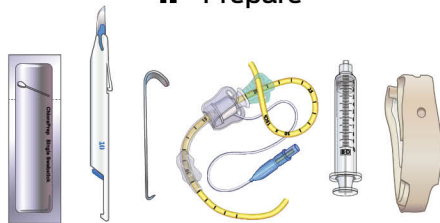


2. Stabilize and locate

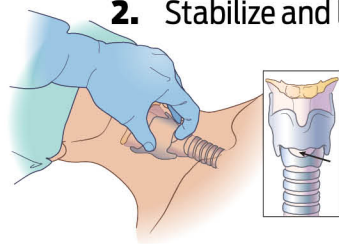


3. Cleanse

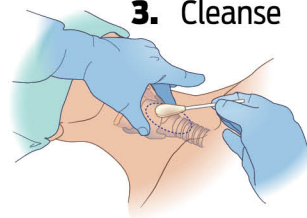


4. Vertically incise

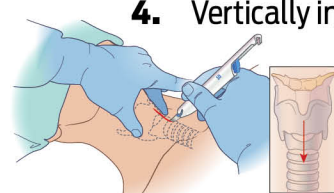

5. Horizontally incise

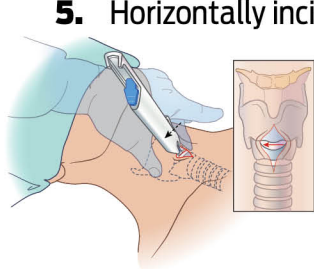


5a. Insert tracheal hook

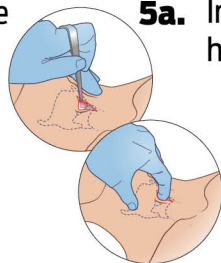


OR


5b. Insert finger

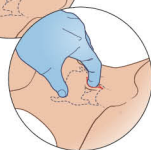


6. Insert bougie

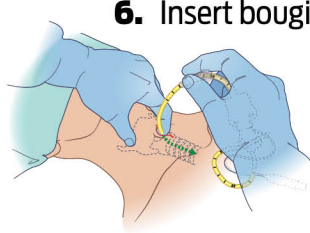

7. Advance tube & inflate cuff

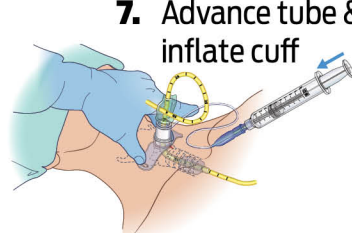

8. Remove bougie

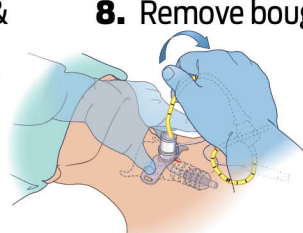

9. Secure tube

