



PN: 50-0039



TELESCOPING COLLAPSIBLE SPLINT

REPACKING TC SPLINT AFTER USE

- 1** Lay the TC Splint out flat in the completely open configuration
- 2** Securely grasp the bar in the open tab end of the TC Splint and push until completely closed
- 3** Secure three hook and loop straps across the splint (starting at the head label of the TC Splint)
- 4** Starting at the head label, begin tightly folding the splint until completely folded and secure the open hook and loop strap across the folded TC Splint.



TELESCOPING COLLAPSIBLE SPLINT

(TC Splint)

Thank you for selecting the
Telescoping Collapsible Splint (TC Splint)
from North American Rescue®

The innovative TC Splint comes complete with 6 (six) rigid, telescoping inner poles – making it easily deployable in the field, but space saving in your pack.

Lightweight and portable, the TC Splint allows for rapid deployment in tactical, field and pre-hospital settings. Size adjusting hook and loop straps, padding and sturdy inner poles help with patient pain and discomfort during transport.



INSTRUCTIONS FOR USE



- 1** Open first strap and begin to unroll/unfold the TC Splint

- 2** Completely unfold, leaving 3 hook and loop straps secured



- 3** Locate and open tabbed end (indicated by red tabs)

- 4** Reach inside the open tabbed end, grasp poles (while securing closed end with other hand) and extend poles to desired length



Instructions Continued on Other Side

INSTRUCTIONS FOR USE • LEG APPLICATION

LEG
5

Place next to patient's injured leg, open the remaining 3 hook and loop straps.



LEG
6

Gently place under the injured leg, following the directional instructions printed on the splint.



LEG
7

Firmly secure all four hook and loop straps around the injured leg.



INSTRUCTIONS FOR USE • ARM APPLICATION

ARM
5

With one or more panels folded in place the splint next to the patient's injured arm, open the remaining hook and loop straps. (Fold panels in for proper fit and length)



ARM
6

Gently place the splint under the injured arm, following the directional instructions printed on the splint.

ARM
7

Firmly secure all four hook and loop straps around the injured arm.



NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277



Manufactured by: North American Rescue, 35 Tedwall Court, Greer, SC 29650, USA