

Raptor® IV Securing Device

REF 30-0006 NSN: 6515-01-523-3317 Qty: 6

Intended Use: Device is intended to secure an IV to a casualty.



Instructions for Use

NOTE: Use only as directed by your EMS authority or under the supervision of a physician.

IV Securing Device

1. Remove white protective film.
2. Place tubing or saline lock on adhesive surface.
3. Fold adhesive surface over the IV tubing or saline lock and secure to the upper extremity.
4. Ensure securing device does not impede arterial or venous blood flow once applied.
5. In the event of device issues refer to medical protocols.
6. Dispose of a contaminated Raptor® per universal protocols.



Made in Taiwan



Lot number



Single use



Medical Device



Consult instructions for use



Manufacturer



Device part number



Date of manufacture



30-0006



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Ct. Greer SC 29650 USA

(01)00842209112098 (11)

(10)



LOT

ZZ-1201

REV060822