

Raptor® IV Securing Device

REF 30-0006 NSN: 6515-01-523-3317

Quantity: 6

Intended Use:

Device is intended to secure an IV to a casualty.

Warning:

Within the European Union (EU) the Raptor® is not indicated for pediatrics and pregnant women due to EU regulatory requirements.

In the event of an incident follow local protocols and report serious incidents to North American Rescue or authorized representative and the competent authority of the member state.



Instructions for Use

NOTE: Use only as directed by your EMS authority or under the supervision of a physician.

IV Securing Device

1. Remove white protective film.
2. Place tubing or saline lock on adhesive surface.
3. Fold adhesive surface over the IV tubing or saline lock and secure to the upper extremity.
4. Ensure securing device does not impede arterial or venous blood flow once applied.
5. In the event of device issues refer to medical protocols.
6. Dispose of a contaminated Raptor® per universal protocols.

(01)00842209112098 (11)

(10)



LOT

ZZ-1201

Raptor® IV Securing Device


REF 30-0006 NSN: 6515-01-523-3317

LOT Lot number


 Manufacturer

 Single use

EC **REP** Authorized Representative

 Do not resterilize

REF Device part number

 Consult instructions for use

 Date of manufacture



Made in Taiwan

MDI Europa GmbH
Langenhagener Str. 71
30855 Hannover-Langenhagen
Germany



REV09102020

 **NORTH AMERICAN RESCUE®**

www.NARescue.com • 888.689.6277

35 Tedwall Ct. Greer SC 29650 USA

ZZ-1201