

KOBUS ROLLUP *The Best of Both Worlds!*

KOHLBRAT & BUNZ CORP

Rollup Litter Convenience + Rigid Carry Capability



**KOBUS RollUP
litter**

**KOBUS RollUP
Bullnose litter**

The **optimal** solution for rescue in confined spaces, heights, depths and horizontal rescues.

The patented RollUP system combines the **advantages** of a roll-up litter with the **rigidness** of a basket carrier.

The Kobus RollUP and Bullnose Litters are designed specifically to optimize the demands required for all aspect of rescue - high angle, confined space, cold weather; marine and fast water applications. Litters are available in 2 different widths: 27.5 and 36.2 inches.

The KOBUS Dura-Slyde (thermo-plastic polymer sliding surface) delivers excellent toughness and strength with a low coefficient of friction. This means no heat generation and outstanding resistance to abrasion and impact during drag maneuvers in a wide range of environmental temperatures and conditions.

KOBUS Litters are designed with 10 stainless steel triangular, permanently affixed lifting / attachment plates with 3 essential grommets per plate for accessories and fixation. Additionally, all Litters feature integrated lifting bridals that make vertical and horizontal hoists a standard capability (no 3rd party straps or adapters needed). Once the patient is secured, the litter becomes rigid. Integrated struts prevent the litter from buckling and prevents pressure being exerted on the patient during the rescue.

KOBUS Litters have been awarded multiple aeromedical certifications for helicopter winching and evacuation operations. All webbing is certified to international standards for lifting, climbing, and flight requirements - including handles and securing straps. All securing straps feature COBRA® steel buckles.

All components have excellent chemical resistance and are easy to clean and decontaminate.

For convenience, the stretcher can be rolled up and stored in an included MOLLE configured backpack and carry case. The carry case also features a removable accessory pouch.

ADDITIONAL SPECS:

- Integrated lifting bridals for vertical and horizontal
- Hoistable. Easy transition between vertical and horizontal hoist
- Fits in confined spaces
- Integrated struts prevent buckling or pressure on the patient
- Multiple aeromedical certifications for helicopter winching/evacuation operations

KOBUS ROLLUP BULLNOSE LITTER

Bullnose design is ideal for drag in snow and undergrowth environments, and provides superior head protection in rescue scenarios

	DESCRIPTION	WIDTH	ITEM #
	KOBUS ROLLUP LITTER GREEN	27.5 In.	60-0104
1	KOBUS ROLLUP LITTER GREEN	36.2 In.	60-0106
	KOBUS ROLLUP LITTER RED	27.5 In.	60-0108
2	KOBUS ROLLUP LITTER RED	36.2 In.	60-0110
	KOBUS ROLLUP BULLNOSE LITTER GREEN	27.5 In.	60-0105
3	KOBUS ROLLUP BULLNOSE LITTER GREEN	36.2 In.	60-0107
	KOBUS ROLLUP BULLNOSE LITTER RED	27.5 In.	60-0109
4	KOBUS ROLLUP BULLNOSE LITTER RED	36.2 In.	60-0111



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

INFLATABLE PILLOW

Kobus presents a compact and durable inflatable pillow which provides stabilization and positioning assistance of the patients head when properly applied. The pillow is added during the RollUp assembly process with no additional time required. It is then inflated when needed, otherwise, it stays out of the way. Inflation time (oral life jacket style inflation) is <15 seconds. As it is inflated the pillow provides cushion to the back of the head while grasping the sides of the head for stabilization. A forehead strap may be applied as needed.

GREEN | 60-0102



RED | 60-0103

TOW STRAP

Kobus Tow Strap with 2 handles & 2 carabiners with adjustable length. Tow Strap with quick release carabiners, adjustable for different pulling setups: single person, two people side by side, or one forward/one aft (requires two straps for ski configuration). Enables efficient use of torso, waist, and arms to reduce fatigue in both single and dual user modes.

BLACK | 60-0114



VERTICAL ANTI-ROTATION STABILIZER (VARS)

Kobus Vertical Anti-Rotation Stabilizer (VARS), when used as instructed the VARS eliminates the need for tag lines when doing 2-man patient/rescuer aeromedical winch evacuations. It is compact, stores in the RollUp carry case, assembles and attaches easily to the 4 pre-positioned Cobra Buckles on the RollUp Litter. The Cobra buckles on the VARS snap into those on the RollUp Litter (prior installation required and remain installed). Control of the litter is attained by moving the VARS in the direction you want the litter to travel thereby preventing spin caused by the downwash from the overhead helicopter blades.

GREEN | 60-0115

RED | 60-0116



FOOT STRAPS

Kobus Foot Strap Set (FSS) for patient positioning and fixing with adjustable length. Eliminates having to create straps out of rope/line which is time consuming and not easily adjusted. The FSS can be secured in less than 3 minutes and when affixed, easily positions and adjusts to the patient minimizing longitudinal movement towards the foot end of the RollUp Litter.

GREEN/BLACK | 60-0100
RED/YELLOW | 60-0101



RED | 60-0099

FLOTATION DEVICE

Kobus flotation device compatible with any RollUp Litter: Highly durable reinforced TPU construction and easy, life jacket style, oral inflation. Installation time is less than minutes each.

GREEN | 60-0098



SHOULDER STRAPS

Kobus Shoulder Straps for patient positioning and fixing with adjustable length. Designed to secure patients at the shoulders and prevent sliding toward the head of the RollUp Litter. These adjustable straps replace time-consuming rope alternatives, can be quickly attached as needed.

GREEN/BLACK | 60-0112

RED/YELLOW | 60-0113



Learn more about
**KOBUS Rollup
Litters** at
www.NARescue.com



NORTH AMERICAN RESCUE

www.NARescue.com • 888.689.6277